

Easy Three-step Interrogation Process

Cell Adapter Must be plugged in and connected here!



1. CONNECT TRANSMITTER






- Ensure connectivity - Included Black Cell Adapter must be plugged into left side of monitor via USB port under rubber flap.
- Plug transmitter into electrical outlet
- Allow several minutes for the transmitter to boot up (LEDs scroll during process)
- Ready to perform an interrogation (only green LED is on)



Turn over for steps 2 and 3 -->

Easy Three-step Interrogation Process

2. INTERROGATE DEVICE

- Position wand over the patient's implanted device
-  Push round START button ONCE to READY transmitter for interrogation
“Stars” icon will light up to indicate the transmitter is ready to perform an interrogation
-  Push button again to initiate interrogation
“Read” icon will flash while the transmitter is locating the implanted device
-  The “Read” icon remains solid while the transmitter LEDs scroll left to right
Data are being collected from the implanted device
-  When the “Tower” icon is lit with scrolling LEDs, you can remove the wand
Data are being uploaded, which may take up to 15 minutes
-  When the “Stars” icon is lit, transmission is complete
“Stars” icon lights up at the start of the process and again at the end



3. CONSULT DEVICE SPECIALIST

Advice on device status and episode interpretation by your Abbott representative.